


# WHY PRICK, WHEN YOU CAN SCAN?\*

If you scan with the **FreeStyle LibreLink<sup>†</sup>** app on your phone, you're almost there.

With the FreeStyle LibreLink app, you **AUTOMATICALLY** have a LibreView account, and your data are **AUTOMATICALLY<sup>†</sup>** stored in the cloud.


There are two easy ways to share your data with your healthcare professional.

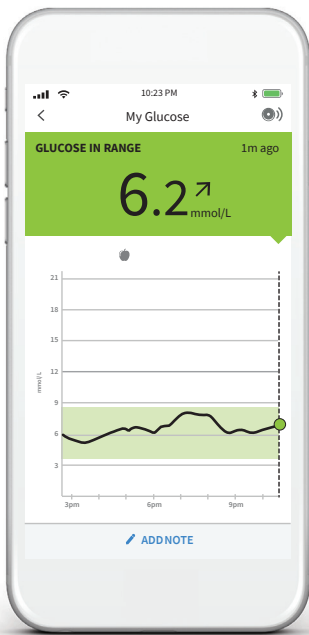

## Option A: OR

- 1** Ask your healthcare professional to **invite you** to share your data.
- 2** You will receive an email inviting you to share. Click **Accept Invitation** and you will be directed to the **LibreView** website.
- 3** Log into your **LibreView** account with the **same email address and password** used to sign up for FreeStyle LibreLink.

## Option B:

- 1** Ask your healthcare professional for his/her **practice ID**.
- 2** Click on the  icon at the top right of the screen and select **Account Settings**.
- 3** Click on **My Practices**.
- 4** Enter your healthcare professional's practice ID.



Need help? Contact Abbott Customer Care at **1-888-205-8296** or visit **www.LibreView.com** and click on **Customer Support**



# Start by getting a LibreView<sup>s</sup> account



FreeStyle  
**Libre**  
FLASH GLUCOSE MONITORING SYSTEM

If you scan with the FreeStyle Libre reader, you need a LibreView account **BEFORE** you can share your data.

- Go to [www.LibreView.com](http://www.LibreView.com)
- Click on **Sign up**
- Choose the **LibreView account for patients** and follow on-screen instructions
- **Connect** your FreeStyle Libre reader
- Press **Upload data** and follow on-screen instructions




There are two easy ways to share your data with your healthcare professional.

## Option A: OR

- 1** Ask your healthcare professional to **invite you** to share your data.
- 2** You will receive an email inviting you to share. Click **Accept Invitation** and you will be directed to the **LibreView** website.
- 3** Log into your **LibreView** account.

## Option B:

- 1** Ask your healthcare professional for his/her **practice ID**.
- 2** Click on the  icon at the top right of the screen and select **Account Settings**.
- 3** Click on **My Practices**.
- 4** Enter your healthcare professional's practice ID.



Need help? Contact Abbott Customer Care at **1-888-205-8296** or visit [www.LibreView.com](http://www.LibreView.com) and click on **Customer Support**



*life. to the fullest.®*

**Abbott**

\* A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycemia or impending hypoglycemia is reported by the system or when symptoms do not match the system readings.

† The FreeStyle LibreLink app and the FreeStyle Libre reader have similar but not identical features. A finger prick test using a blood glucose meter is required during times of rapidly changing glucose levels when interstitial fluid glucose levels may not accurately reflect blood glucose levels or if hypoglycemia or impending hypoglycemia is reported by the FreeStyle LibreLink app or when symptoms do not match the app readings. The FreeStyle Libre sensor communicates with the FreeStyle Libre reader that started it or the FreeStyle LibreLink app that started it. A sensor started by the FreeStyle Libre reader will also communicate with the FreeStyle LibreLink app. The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check the website for more information about device compatibility before using the app. Use of FreeStyle LibreLink requires registration with LibreView.

‡ Use of FreeStyle LibreLink requires registration with LibreView. Automatic upload requires a wireless internet connection or mobile data connection.

§ The LibreView website is only compatible with certain operating systems and browsers. Please check [www.LibreView.com](http://www.LibreView.com) for additional information.

© 2020 Abbott. FreeStyle, Libre, and related brand marks are trademarks of Abbott Diabetes Care Inc. in various jurisdictions. Product images are for illustrative purposes only.

ADC-12207 v2.0